

Emergency Caller Products

GETS
HELP
FAST!


(T) 800-227-2474 (F) 800-762-7613

AT-900 Automatic Mandown Supervised Wireless Transmitter

Note* For use with
Emergency Caller Products
900MHz Receiver Products



- 1 Mandown Transmitter with
Necklace.
When activated the transmitter
will send a signal to the
Receiver.

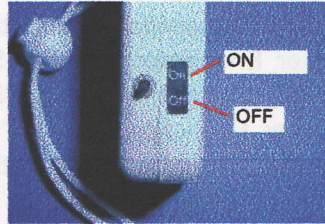

- 2 The Mandown transmitter has
an On/Off switch on the side
to activate or deactivate the
transmitter unit. Be Sure to
switch to "OFF"



- 3 The transmitter code is marked
on the unit and is preset at the
factory. There is the capability
to change the code, If used
with the EM-900 Monitoring
Station or ERU-900 PERS.



- 4 The AT-900 may be worn on the
necklace and detects resident
Fall at any angle. No false
alarms from bending over, swing-
ing necklace, etc.
Important! Be sure to test your
system often, with transmitter at
locations where patient may be.



- 5 The AT-900 may also be pinned to cloth-
ing above the waist. Such as on clothing,
inside shirt pocket, outside or inside
sweater or coat. The transmitter must be
secured so that it can not fall to a hori-
zontal (flat) position which would activate the alarm.
Important! Be sure to turn your transmit-
ter "OFF" whenever you are not upright or
laying transmitter down, to avoid false
alarms.



- 6 Another way to wear the AT-900
is attached to a belt clip. Be sure
to attach it so the unit is upright.

The belt clip should be
adjustable to maintain upright
position of transmitter.
Belt clips are available at Pager
and electronic supply stores.

Additional ECP Products that operate with the Automatic Man Down Transmitter.

MD-900-Care Caller-Enunciator with Automatic Mandown Transmitter and necklace.

Em-900 -40 patient monitoring System- includes Monitoring unit, Charge cradle and AC adapter.

ERU-900- Emergency Response Unit; Receiver/Dialer Pendant Transmitter-wrist band and necklace.

ER-900 - Additional Emergency Response unit.

LR-900 - Intelligent Repeater for Extending Range.

The Mandown AUTOMATIC TRANSMITTER sends the signal to call for help up to 1000 feet indoors (depending on structure, metal obstacles etc.). Be sure to test your system often as range may be effected by conditions.

See Reverse side for a list of other ECP Products.

Order Information: Ready-to-Go Complete Systems or Customize your own

OTHER LONG RANGE FALL PROTECTION SYSTEMS AVAILABLE FROM ECP.

Fall Alert Complete Systems - 900 MHZ Long Range - includes accessories

- FA-900-1 Bed/Rail System with LT-900 transmitter, Enunciator, 1" x 30" Sensor Strip Pad, AC Adapter
- FA-900-2 Bed/Rail System with LT-900 transmitter, Enunciator, 2" x 30" Sensor Strip Pad, AC Adapter
- FA-900-3 Wheelchair-Seat System with LT-900 transmitter, Enunciator, 9"x16" Sensor Pad, AC Adapter
- FA-900-4 Bed System with LT-900 transmitter, Enunciator, 14" x 24" Sensor Pad, AC Adapter
- FA-900-5 Bed System with LT-900 transmitter, Enunciator, 18" x 24" Sensor Pad, AC Adapter
- FA-900-6 Floor System with LT-900 transmitter, Enunciator, 24" x 30" Sensor Pad, AC Adapter
- FA-900-7 Floor System with LT-900 transmitter, Enunciator, 24" x 36" Sensor Pad, AC Adapter

ALL ECP wireless 900 MHz transmitters have Supervision feature. Every 60 seconds the transmitter sends an 'I'm Here' and a "Transmitter Battery" signal to the receiver. If the signal is not sensed by the receiver in a thirty minute period, the receiver will sound it's local alarm. This means protection from low transmitter battery, non-working transmitters or transmitters removed from the facility.


Emergency Caller Products presents advanced technology products to address those who need dependable care!

Emergency Response



You will be very happy with the NEW ERU-900. Whenever there's an Emergency, a quick press of the Pendant (wear it on your wrist, neck, or handheld) sends the signal to "Get Help Fast!" The Caller is triggered to deliver the emergency message! The Pendant is water resistant and can even be worn in the shower or bathtub! There are so many wonderful features with the ERU-900. Especially **No Monitoring Fees!**

Fall Prevention

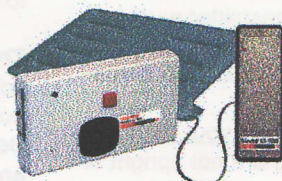

Wireless Fall Prevention Systems and Products including New Automatic "Man Down" transmitters that eliminate push buttons and pull cords. Also models with Universal transmitter that feature "pause", "sound alarm and transmit signal" when activated. Transmitters operate with Lifetime Sensor Pads and other sensors. Wireless Receiver Controllers include local PERS units, Enunciators and 40+ patient Monitor Station.

Wanderer Alert



WORKS AUTOMATICALLY! Detects patient wandering away and then sounds the alarm and calls for help! The Pendant Transmitter automatically sends a signal every 10 seconds to the Receiver-Dialer. If the receiver doesn't hear from the pendant within a 3 minute period, (indicating resident has traveled a distance away) the receiver-dialer will automatically sound the Alarm Strobe and/or dial emergency numbers and deliver the emergency message. **No Monitoring Fees!**

Lifetime Sensor Pads



Lifetime Sensor Pads are available in many styles. Wheelchair (seat), bed, and floor. Exclusive Sensor Strip for beds and rails, allow for total comfort and freedom of movement in bed. Pads are thin and resident comfortable. Sensor pads are slip resistant and Completely Waterproof! Standard with velcro mounting tabs and 8 ft lead (chair-seat pads with 4 ft lead). Pads quick lock connect to transmitters and devices with RJ-22 plug. Lifetime Sensor Pads **DIRECTLY REPLACE all disposable sensor pads.** With Lifetime Guarantees, you'll never again need constant replacement with expensive costs!

Emergency Caller Products

Emergency Caller Products (T) 800-227-2474 (F) 800-762-7613

Made in USA