

# Emergency Caller Products GETS HELP EAST!

## P-6-22

### Wheelchair-Seat Sensor Pad 9" x 16"

#### Mounts on Wheelchair & Chairs

#### USER MANUAL

#### QUICK START GUIDE



1. Remove Pad from shipping bag. Remove cord from plastic covering.



2. Uncoil the cord from the sensor pad, removing any slack. Place sensor pad on the wheelchair with the cord coming out behind the back of the chair. **For best results place the mat horizontally in chair with LABEL SIDE UP. Do Not Place sensor pad under cushions.**



3. Extend the uncoiled cord towards the back of the wheelchair. Now you are ready to attach the connector to any alarm device, preferably Wireless Universal Transmitters or Patient Alert (PA) from ECP. Operates with most mfrs. devices.

The RJ22 connection at the end of the cord may be connected into any of Emergency Caller Products alarm units such as the Patient Alert, the Extended Range Transmitter, TR-300, or the Long Range Transmitter, the LT-900. Consult your Emergency Caller Products sales representative to learn how these other products work with this model sensor pad. The pad may also be connected into other products that will receive the RJ22 connector.

Sensor Pads detect both applied weight "Patient On" pad or removed weight "Patient Off". Selection made by switch on Universal Transmitter Unit or other monitoring products.

Sensor pads are completely waterproof. Lifetime sensor pads replace all disposable sensor pads. Emergency Caller Sensor pads have been used for over 25 years in security applications. They are durable, safe and withstand both outdoor and indoor environments. With lifetime guarantee you never again need to be concerned about constant replacement and expensive costs

**Emergency Caller Products (T) 800-227-2474 (F) 800-762-7613**

www.emergencycaller.com sales@emergencycaller.com

U.S.A. and World Patents Issued and Pending on Certain Products

